




PRODUCT INSTRUCTION SHEET

PRODUCT CATEGORY: Sensonics' Microencapsulated Smell Tests

PRODUCT NAME: Brief Smell Identification Test™ (B-SIT®)

DESCRIPTION:

This 12-item version of the Smell Identification Test®, also known as the Cross-Cultural Smell Identification Test, is useful for quantifying smell loss in situations where less than five minutes of time are available. This test is useful in population surveys and general workplace studies. The odors are well known in most cultures. Percentile norms, based upon thousands of subjects, are available within The Brief Smell Identification Test™ Administration Manual. If this is the first time ordering, we recommend purchasing this manual.

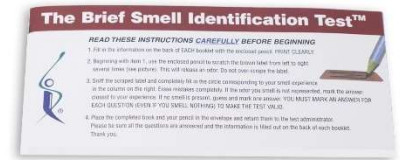

INSTRUCTIONS:


- Fill in the information on the back of EACH booklet with the enclosed pencil. PRINT CLEARLY.
- Beginning with item 1 of the booklet, use the enclosed pencil to scratch the brown label from left to right several times (see picture). This will release an odor. Do not over-scrape the label.
- Sniff the scraped label and completely fill in the circle corresponding to your smell experience in the column on the right. Erase mistakes completely. If the odor you smell is not represented, mark the answer closest to your experience. If no smell is present, guess and mark one answer. YOU MUST MARK AN ANSWER FOR EACH QUESTION (EVEN IF YOU SMELL NOTHING) TO MAKE THE TEST VALID.
- Place the completed book and your pencil in the envelope and return them to the test administrator. Please be sure all questions are answered, and the information is filled out on the back of the booklet. Thank you.



Each test is single use only



Keep Dry

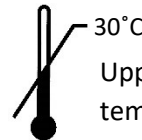

30°C

Upper limit storage temperature



Sensonics International
 411 South Black Horse Pike
 Haddon Heights, NJ 08035
 United States
 Contact no. (856)-547-7702
 MANUFACTURER www.sensonics.com



MedEnvoy Global B.V.
 Prinses Margrietplantsoen 33 - Suite 123
 2595 AM The Hague
 The Netherlands