



PRODUCT INSTRUCTION SHEET

PRODUCT CATEGORY: Sensonics' Microencapsulated Smell Tests (G051SITTK)

PRODUCT NAME: The Pediatric Smell Wheel™


DESCRIPTION:

The Pediatric Smell Wheel™ is an attractive game-like test for evaluating olfaction in children. A rotating cardboard disk exposes one of 11 odors at a time for sampling. Multiple-choice alternatives use both words and pictures. When the disk is completely rotated, the child's answers appear as dark dots in a series of holes in the jacket. The number of dark dots signifies the test score. Can be used as an educational game as well as a smell test. Ideal for classroom education settings.



INSTRUCTIONS:


- Turn the wheel clockwise, starting at odor #1.
- Scratch and sniff the brown labels. Fill in your answers in the circles.
- Stop at the stop sign. Do your answers appear through the eleven small holes on the front cover?
- See back for your score, based on the number of correct responses.

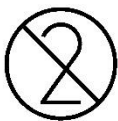

Each test is single use only



Keep Dry

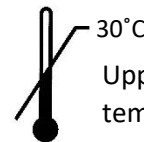

Upper limit storage temperature



Sensonics International
411 South Black Horse Pike
Haddon Heights, NJ 08035
United States
Contact no. (856)-547-7702

MANUFACTURER www.sensonics.com

EC

REP

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands