



PRODUCT INSTRUCTION SHEET

PRODUCT CATEGORY: Sensonics' Microencapsulated Smell Tests (G051SITTK)

PRODUCT NAME: The Pocket Smell Test™ (PST)

DESCRIPTION:

The Pocket Smell Test™ provides a quick, forced-choice means of testing Cranial Nerve I function. To fully characterize an olfactory problem discovered by this test, administer the 40-item Smell identification Test™. The latter test provides norms for both sexes of all ages, clear-cut diagnostic categories, and means for detecting malingering



INSTRUCTIONS:

- Release first odor with tip of pencil or pen.
- Read response alternatives to patient and circle patient's answer.
- If desired, test each nostril separately.
- Repeat with remaining two odors.
- One or more incorrect responses suggest olfactory dysfunction. To fully characterize the problem, administer the 40-item Smell Identification Test™



Each Pocket test is single use only



Keep Dry



30°C

Upper limit storage temperature



MANUFACTURER

Sensonics International
411 South Black Horse Pike
Haddon Heights, NJ 08035
United States
Contact no. (856)-547-7702
www.sensonics.com

EC

REP

MedEnvoy Global B.V.

Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands