



PRODUCT INSTRUCTION SHEET

PRODUCT CATEGORY: Sensonics' Microencapsulated Smell Tests (G051SITTK)

PRODUCT NAME: The Quick Smell Identification Test™ (Q-SIT®)


DESCRIPTION:

The Quick Smell Identification Test™ or Q-SIT® is a rapid assessment of smell ability for bedside/medical applications. The Q-SIT® comes in convenient booklets with a tear-off portion for presentation to the patient. The booklet contains 20 tests.



INSTRUCTIONS:

- Detach the end of the page containing the three scents along the perforation on the right.
- Present the detached end to the patient. Beginning with the top scratch 'n sniff label (A), ask the patient to scratch the surface with his or her fingernail, sniff the odor, and select the best answer from the choices provided.
- Record the response for each scent in the appropriate box area. The correct answer is indicated in bold. The score is the number of scents correctly identified out of three.
- If the patient misses one or more items, administer Sensonics International's 40-item Smell Identification Test™ to assess the degree of smell loss (www.sensonics.com).

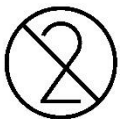

Each test is single use only



Keep Dry



30°C

Upper limit storage temperature



MANUFACTURER

Sensonics International
411 South Black Horse Pike
Haddon Heights, NJ 08035
United States
Contact no. (856)-547-7702
www.sensonics.com

EC

REP

MedEnvoy Global B.V.

Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands